




Green sampling stick and tube for taking sample

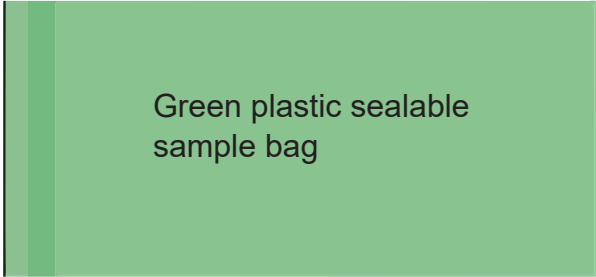

Green plastic sealable sample bag

- 1

Collect the sample by scraping the sampling stick on your bowel motion (stool) until the end of the stick is covered.
- 2

Put the green sampling stick back into the tube. Twist the green cap and push until you hear a click. Do not reopen the tube. Wash and dry your hands.
- 3

Put the green collection tube into the green plastic bag. Close the green plastic bag with collection tube inside.

To be completed by the patient

- 4

Please complete the test request form on ORCIS ensuring all required information is completed. Print form, Label samples, Issue to Lab.

- 5

Place the green bag containing the sample tube inside the sealable section of the sample transport bag, and seal. Place the completed request form into the form pocket of the sample transport bag. (Sample and request form must not be packed in the same section of the sample transport bag).

- 6

Sample can be sent with bloods, and other chemistry samples to:

Eurofins Biomnis
 Unit 3, Sandyford Business Centre,
 Sandyford Business Park,
 Blackthorn Road,
 DUBLIN 18
 D18 E528

Not suitable for C&S testing, for FIT tests only.