

ORDERS

orders@CTIE.eurofinseu.com

CLIENT SERVICES DEPT. (Results and Test Enquiries)

clientservices@ctie.eurofinseu.com
1800 252 966

EUROFINS LABLINK (Specimen Transport Services)

lablinklogistics@ctie.eurofinseu.com
1800 252 967

ORCIS PORTAL (General Queries)

portal@ctie.eurofinseu.com

ORDER OF DRAW	TUBE CONTENTS	TUBE COLOUR	SPEC VOL.	MIXING INSTRUCTIONS	ASSAY	SPECIAL INSTRUCTIONS	CAT. NUMBER
1	Z DISCARD TUBE	 Red Black	5mL	Not for analysis and used to prime the line	Not for analysis and used to prime the line	<ul style="list-style-type: none"> • Prior to collecting blood samples from a newly inserted peripheral venus cannula (PVC) • Prior to collecting a coagulation sample using a butterfly needle • Prior to collecting blood samples from a central line 	456202
2	COAGULATION SODIUM CITRATE	 Blue Black	3mL	After Blood Collection Invert tube 4-5 times	Coagulation Studies	<ul style="list-style-type: none"> • COAG must be filled to the fill mark • Inadequately filled tubes cannot be used 	454334
3	Z SERUM CLOT ACTIVATOR	 Yellow Red	5mL	After Blood Collection Invert tube 5-10 times	General Biochemistry, Immunology Tests, Virology Tests, B12, Folate, Ferritin and Drug Level Assays.	<ul style="list-style-type: none"> • Fill to fill mark 	456071
4	LH LITHIUM HEPARIN SEP	 Yellow Green	3.5mL	After Blood Collection Invert tube 5-10 times	General Biochemistry, Immunology Tests, Virology Tests and Drug Level Assays.	<ul style="list-style-type: none"> • For use in Haematology Oncology day care area only • Use with patients with clotting disorders 	454454
5	SODIUM HEPARIN	 Black Royal-blue	6mL	After Blood Collection Invert tube 5-10 times	Trace Metal Analysis	<ul style="list-style-type: none"> • Trace metal to be taken only when specifically requested 	456080
6	EDTA	 Black Lavender	3mL	After Blood Collection Invert tube 5-10 times	FBC, ESR, Red Cell Folate	<ul style="list-style-type: none"> • Fill to fill mark 	454023
7	EDTA CROSSMATCH	 Black Pink	6mL	After Blood Collection Invert tube 5-10 times	Cross Match, Blood Transfusion Tests	<ul style="list-style-type: none"> • Label must be handwritten with Full Name, DOB, Chart Number and Drawers Signature 	456093
8	SODIUM FLUORIDE / POTASSIUM OXALATE	 Black Grey	4mL	After Blood Collection Invert tube 5-10 times	Glucose, GTT	<ul style="list-style-type: none"> • Specify collection time & state if sample is random or fasting 	454297